

YOUR HOT TIP!!

Essentials: Different functions of the PIPA pressure mattress.



NORMAL ALTERNATING AIR: Most common setting for patients on a pressure-relieving mattress. Pressure Area Cares (PAC) still required, just not as often as on a standard mattress.



GENTLE / LIGHT ALTERNATING MODE: Great for the frail and elderly or find the normal mode uncomfortable. PAC still required.



STATIC / NON-ALTERNATING MODE: This a would feel like a normal mattress and the patient would need standard PAC as for a standard mattress.



MAX FIRM: This will inflate all sections of the mattress to maximum. This is ideal for patient cares and transfers. The mattress will revert to its previous mode after 20mins.